## DE

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Art. 2007

## EN

The suprima hip protector offers an effective protection against hip fractures and injuries in the event of a fall. The pant has protector pockets on the right and left side which are exactly positioned to hold the protectors.

- our protectors are certified according to the European test standard CE EN 1621-2.
- The affecting force is slowed and weakened.
- The impact energy on the thigh neck is reduced by the protector and distributed to the surrounding soft tissue.
- Viscoelastic material returns to its original state after a fall and protects immediately.
- Protectors are lightweight and comfortable to wear. Acceptance in the patient is very high (demonstrated by case study).

### Hip protectors made of viscoelastic soft foam

#### application (article with changing system)

Before putting on the suprima panty, put the protectors with the wide side down in the two-sided pockets (the side with the punch to the body) and put the inner fabric over the protector. Thus, this is at the fall-prone place, the femoral neck, fixed and can not slip. (sewn in art. 1424, 1481, 1491)

#### cleaning (item no. 2007)

For cleaning, remove the protectors from the pockets and wash them gently with a commercial detergent at a maximum of 40 ° C by hand.

#### cleaning (item no. 2008)

The protectors are welded in waterproof foil. For cleaning, the protectors can be washed up to 60 ° C in the washing machine. Please do not use fabric softener and do not put in the dryer.

#### Attention

Hip protectors can not prevent injury in every case.  
Any liability is therefore excluded.

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